




PART 1 – PREPARE DISPOSABLE APPLICATOR DEVICE AND APPLICATOR UNIT

Turn ON Applicator Unit on the back and connect foot switch




1. Place 21g de pH 10 vial, draw fluid into 1 ml syringe

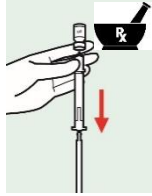


2. Prepare Co-D. solution to be co-applied with PRF




3. Transfer disposable applicator + pH-paper to sterile area

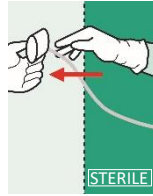

4. Transfer syringe holder out of sterile area

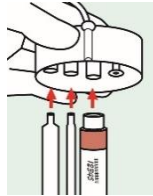


5. Insert PRF, pH 10 and Co-D. solution syringes into syringe holder

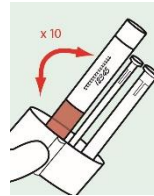


6. Rotate syringe holder back and forth (x10)




7. When indicated in display open shield

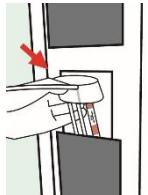

8. Insert and position syringe holder

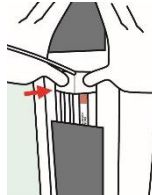

9. Press syringe holder into cavity using two fingers

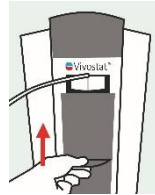


10. Close shield

CAUTION:

- 1) To avoid air bubbles impacting mixture negatively do NOT push needle to the end of pH 10 vial in Step 1
- 2) Follow Step 6 carefully for platelets not to settle at the bottom, as they are essential in the healing process
- 3) If +30 min. pass after step 6 rotate syringe holder again: Press button on top of Applicator Unit for +4 sec., take out syringe holder, rotate it, push syringe holder into reception cavity, close shield and press button for +4 sec.


PART 2 – LOAD SYRINGES, PRIME AND INITIATE APPLICATION

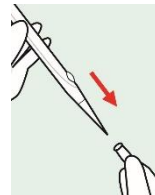


1. Take cap off Spraypen




2. Press button to start loading syringes

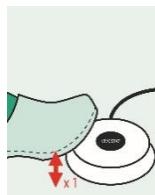

3. Press foot switch once to prime

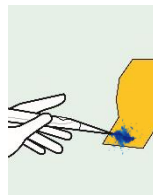


4. Point to lower corner of pH-paper while auto-priming

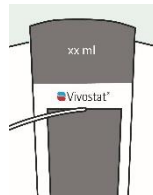

5. Auto-priming stops automatically and ml + spray mode is displayed




6. Verify correct mixture on pH-paper (from liquid to viscous in 10-30 sec.)

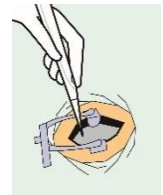

7. Ready for application


ADDITIONAL APPLICATION TIPS

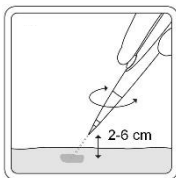


Use circular movements in a 2-6 cm spray distance




Do NOT poke tissue and bend Spraypen tip

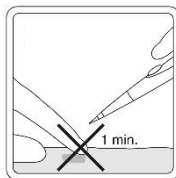

Avoid touching the applied substance immediately after application

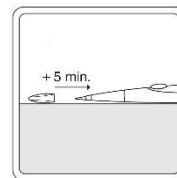

If pausing application +5 minutes, put cap on Spraypen

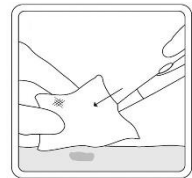

Remove cap and wipe tip of Spraypen prior to reapplying